

Ventrain®

Redefining
Ventilation



Airway
Innovation



Ventrain®

Flow Control Ventilation (FCV®)

Ventrain is a manual ventilator that allows complete ventilation by active control of both inspiratory and expiratory flow through a small diameter lumen (2 mm).

This revolutionary technology, which for the first time allows complete control of the entire ventilatory cycle, is called **Flow Control Ventilation (FCV®)**.

It combines a source of oxygen that generates the pressure necessary to overcome the inspiratory flow resistance of a small-bore airway with sufficient subatmospheric pressure to facilitate active expiration.

Traditional alternatives to small-bore catheter ventilation (e.g., jet ventilation) are inherently dangerous if the airway is not sufficiently open. Insufficient exhalation of gas during expiration will lead to auto-PEEP, with deleterious consequences on the patient's hemodynamic status.

If ventilation is maintained it can lead to an increase in intrathoracic pressure within a few cycles, resulting in barotrauma and/or circulatory collapse.

Unlike manual high-pressure (jet ventilation) ventilators, in which inspiratory pressure is adjusted but inspiratory volume can hardly be estimated, **Ventrain®** is a flow-controlled device, so inspiratory tidal volumes can be easily estimated from the adjusted flow and inspiratory time [Table 1].

Ventrain® in combination with a Cricath trans-tracheal catheter or a **Tritube®** ultrafine endotracheal tube is the device of choice in the management of the emergent clinical airway-cricothyroidotomy "Non Ventilable Non Intubable" (NVNI) or elective process; difficult airway, pediatric airway, upper airway surgery, tracheal, with bronchial blockers or through the working channel of flexible or rigid bronchoscopes.

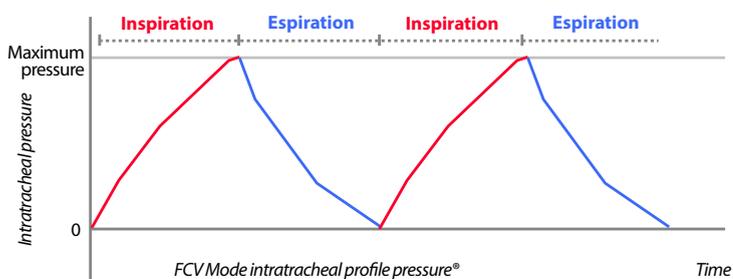



TABLE 1 - Flow rate and volume administered

Flow (Liter/min)	Volume administered (ml) in 1 sec of inspiration.	Volume administered (ml) in 2 sec of inspiration.
2	33	66
3	50	100
6	100	200
9	150	300
12	200	400
15	250	500

Tritube®

Ultra-thin endotracheal tube

ID = 2.4 mm Ø
OD = 4.4 mm Ø

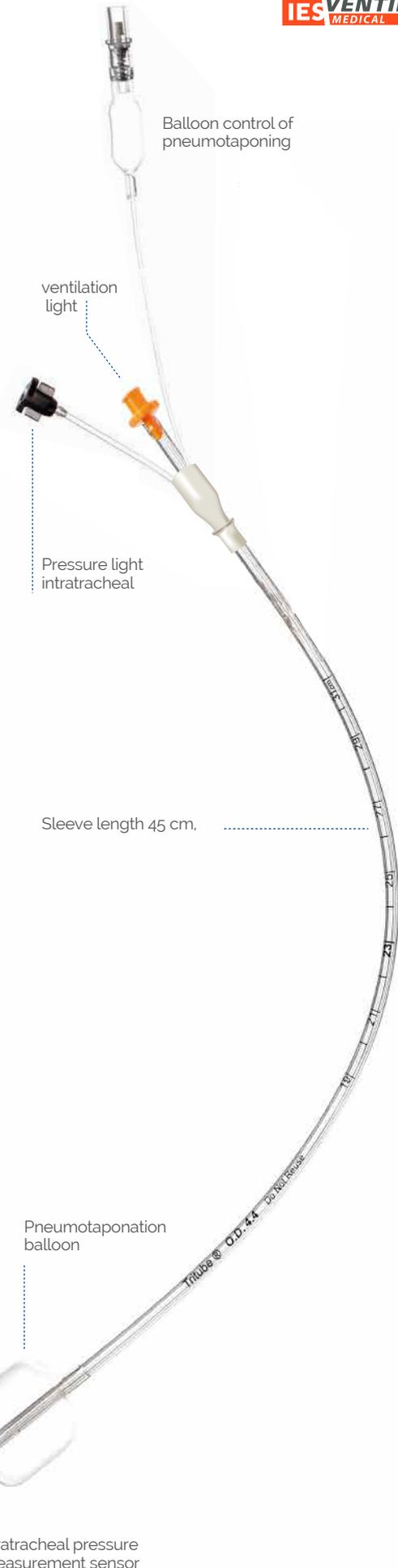

Cricath®

Cricothyroidotomy catheter

ID = 2 mm Ø

Luer lock for connection to venting devices

Anatomically curved needle for ease of insertion

Resistant and flexible 70 mm catheter made of high quality Teflon.

Anti-tilting mechanism that reduces the risk of kinking



Connection to other devices



INDICATIONS

In emergencies,

«Non Ventilable, Non Intubable » (NVNI)



Non Ventilable Non Intubable* patients represent an emergency that may require immediate access to the airway by a cricothyroidotomy technique.

Ventrain® in combination with the Cricath® Transtracheal Catheter is the first and only emergency ventilation device on the market that uses a 2 mm internal diameter catheter inserted through the cricothyroid membrane:

- Allows control of the entire ventilatory cycle (inspiration and expiration) by means of **FCV®** technology.
- It offers a minute volume (MV) greater than 6 L/min and an I/E ratio of 1:1, when using a flow rate of 15L/min.
- Cricath, the smart choice.
 - Anti-tilting mechanism: reduces the risk of kinking
 - Small caliber: minimally invasive
 - Curved needle: easy insertion.

The **Ventrain®** Emergency Kit is the solution in emergency airway situations. It includes the **Ventrain®** ventilation device and the **Cricath®** cricothyroidotomy catheter.



Full video demonstration
Cricath

<https://youtu.be/TyeTtwL0Jo>





INDICATIONS

Upper airway surgery

Ventrain® ventilation during upper airway surgery combines the physical advantages of using a small-bore catheter with the benefits of **Ventrain®** ventilation:

Improves the vision of the surgical field, facilitating surgery and reducing its duration.

- Complete ventilation (oxygenation and CO₂ elimination)
- Minute volume up to 7 l/min.
- No vibration of the vocal cords.
- Minimally invasive.
- Increases the surgical field
- Better vision for the surgeon.



Examples of difficult airway access with Ventrain, Tritube and

Use of the **Tritube**: a considerable improvement of the vision of the surgical field during access to the difficult airway is appreciated.




YouTube : Tritube video
<https://goo.gl/fY1RHn>

Patient with airway obstruction due to a polyp. Achieves complete ventilation during the procedure thanks to the use of **Ventrain** and **Cricath**.



YouTube : Ventrain polyp
<https://goo.gl/ud8Fv1>

Complete ventilation with Ventrain during obstructed airway surgery

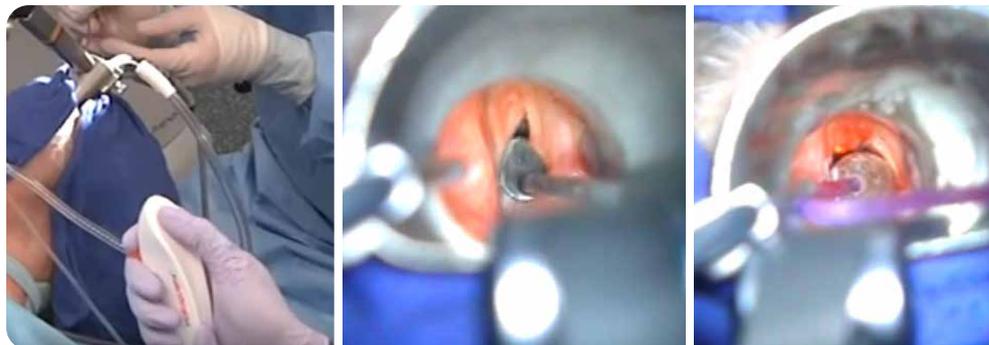

YouTube : Ventrain
<https://goo.gl/8eQFUS>

INDICATIONS

Selective pulmonary ventilation

The surgeon requires an adequate surgical field to perform optimal surgery. One-lung isolation techniques have been developed to increase this crucial working space by collapsing one lung and ventilating the other.


Complications associated with this ventilation include increased risk of hypoxemia due to impaired ventilation-perfusion and dissociation of O₂ from hemoglobin (Bohr effect).

Oxygenation of the collapsed lung may require the use of continuous positive airway pressure (CPAP). However, this is not always sufficient and may result in unwanted re-expansion of the lung.


In addition, with **FCV**[®] ventilation, collapse of the non-dependent lung is achieved more quickly and maintained more effectively throughout the surgery, preventing the procedure from being delayed or prolonged.

Ventrain[®] can be used in conjunction with a double-lumen tube or bronchial blocker during single-lung isolation procedures.

Ventrain[®] is extremely useful when additional ventilation through a double-lumen tube is required.

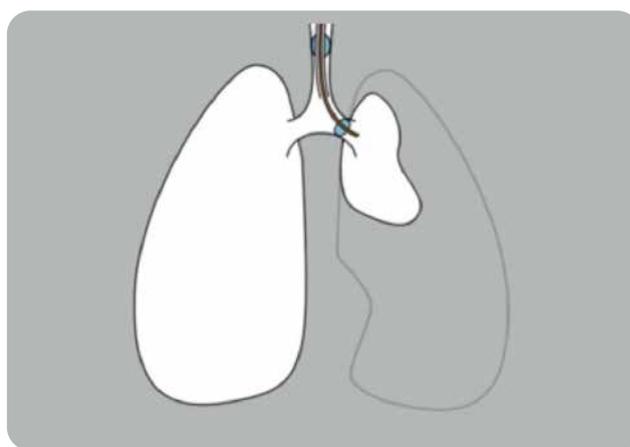

Ventrain[®] together with a bronchial blocker

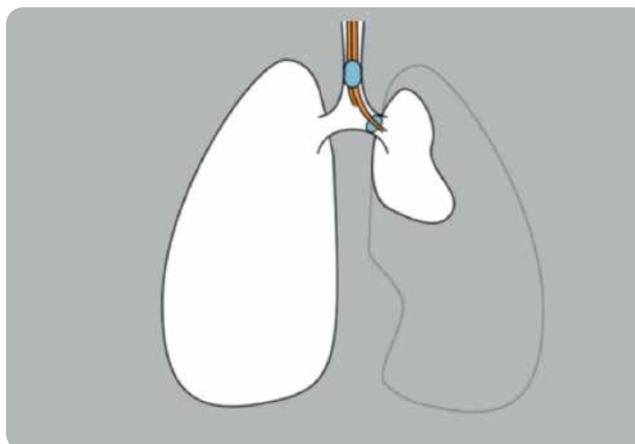

Ventrain[®] together with double light tube

A small endotracheal tube (e.g., tube exchange catheter or intubation introducer; ID 2-3 mm) is inserted into the bronchial lumen of the double-lumen tube, connecting it to the **Ventrain®**.

Moderate flow ventilation (2-6 L/min; 100% O₂) will prevent or resolve hypoxemia, while the lung remains collapsed.

If a lung needs to be collapsed in a patient who already has breathing difficulties, a bronchial blocker can be used to isolate the lung. The **FCV®** technology used in **Ventrain®** will allow complete ventilation through the small bore blockers. The non-dependent lung will remain collapsed with this method of ventilation.


Advantages of Ventrain® in lung isolation techniques:

- Complete ventilation (oxygenation and active CO₂ removal)
- No re-expansion of non-dependent lung.
- Minimally invasive insertion
- Radically shortened lung collapse time.
- Good mobilization of bronchial secretions.
- Improved vision of the surgical field facilitating surgery and decreasing surgery time.



How does Ventrain® work?

Figure 1 shows how the narrow area of the nozzle increases the gas velocity creating a negative pressure or Venturi effect (Bernoulli's Law) when orifice 2 is closed and orifice 1 is open.

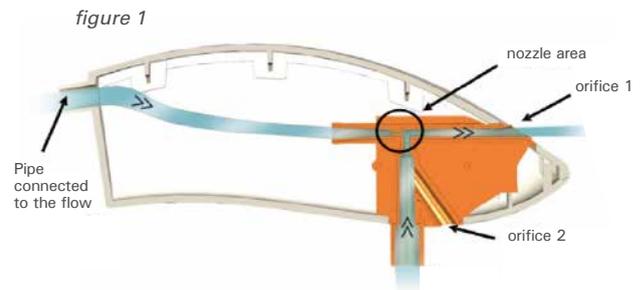

How is Ventrain® used?

Inspiration

Close orifices 1 and 2. Oxygen flows into the lungs.

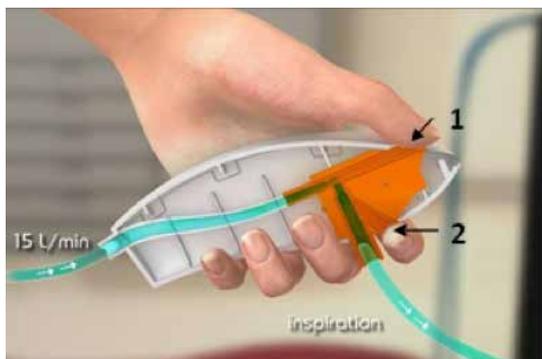


Figure A3a: Ventrain in inspiration position

Expiration

Open orifice 1 and close orifice 2. The gas will be actively aspirated from the lungs.

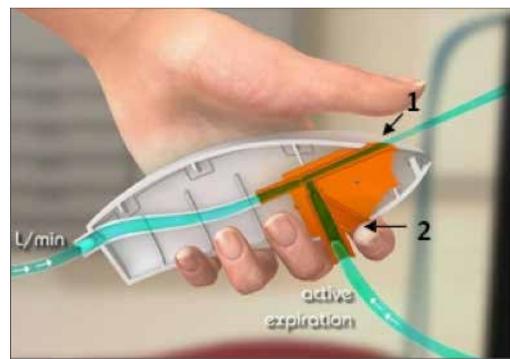


Figure A3b: Ventrain in exhalation position .



What happens when Ventrain® orifices 1 and 2 are kept open?

This is called the equilibrium position. When orifices 1 and 2 are held open, there is (almost) no gas entering the lungs.

The equilibrium position can be used for two purposes:

- On / off button:

The equilibrium position functions as an "on / off button". When orifice 2 (no matter what is being done with orifice 1) is open, there is no inspiration and no active expiration.

- Bring the positive or negative pressure in the lungs back to zero:

This so-called equilibrium position can also be used if the user does not see clear chest excursions (upward and downward movement of the chest) or has doubts about the current pressure in the lungs. When hole 2 is held open for at least 5 seconds (no matter what is being done with hole 1), the pressure in the lungs equilibrates with the pressure in the environment:

³ There is only a slight positive pressure of ~ 2.3 mbar*1, but not effective enough to create significant inspiration. For more details, see "Ventrain: an ejector for emergency use", AEW Hamaekers et al, Department of Anesthesia, MUMC.

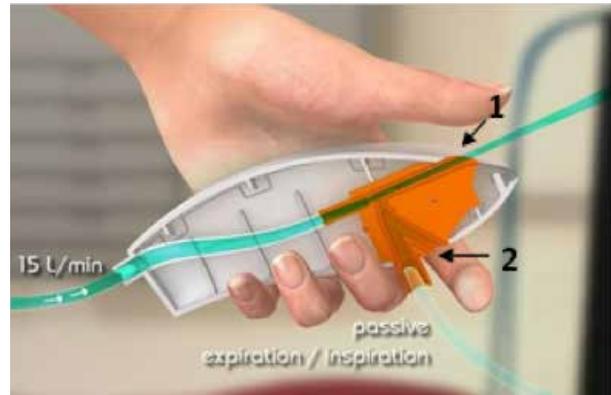


Figure A4: Ventrain in equilibrium position / "off" position

There will only be a minimal negative pressure of ~ 6.5 mbar, but it is not effective enough to generate an expiration. For more details see Ventrain: an ejector for emergency use, AEW Hamaekers et al, Department of Anesthesia, MUMC.

What happens when only orifice 1 is closed?

When only orifice 1 is closed (almost ³) no flow goes into the lungs and no air is actively sucked out of the lungs.

During this position **Ventrain®** makes a much louder noise (easily recognizable) and does not work (off position).

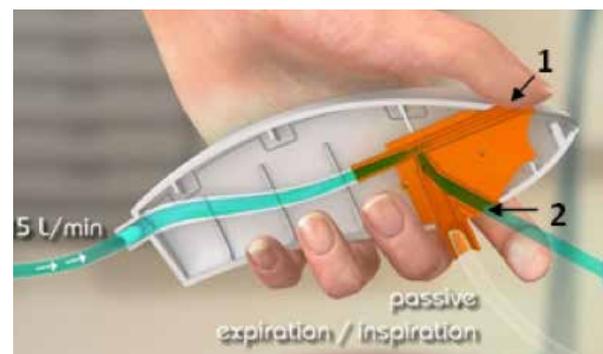


Figure A5: Ventrain "off" position

Ventrain®

Redefining Ventilation



BIBLIOGRAPHY

- Blunt MC, Young PJ, Patil A and Haddock A. Gel lubrication of the tracheal tube cuff reduces pulmonary aspiration. *Anesthesiology*. 2001 Aug;95(2):377-81
- Dullenkopf A, Gerber A and Weiss M. Fluid leakage past tracheal tube cuffs: evaluation of the new Microcuff endotracheal tube. *Intensive Care Med*. 2003 Oct;29(10):1849-53
- Sengupta P, Sessler DI, Maglinger P, Wells S, Vogt A, Durrani J and Wadhwa A. Endotracheal tube cuff pressure in three hospitals, and the volume required to produce an appropriate cuff pressure. *BMC Anesthesiol*. 2004 Nov 29;4(1):8
- Spiegel JE. Endotracheal tube cuffs: Design and function. *Anesthesia News Guide Airway Management* 2010
- Upper Airway Surgery:**
 - M.S. Kristensen, M.W.P. de Wolf and L.S. Rasmussen. Ventilation via the 2.4 mm internal diameter Tritube with cuff – New possibilities in airway management. *Acta Anesthesiologica Scandinavica* Apr 2017 epub
 - Borg et al. Ventrain for ventilation of the lungs. *Br J Anaesth*. 2012 Nov
 - Fearnley et al. Elective use of the Ventrain for upper airway obstruction during high frequency jet ventilation. *J Clin Case Rep* 2016
 - Monnier et al. Transthyroid access to the larynx for endoscopic resection of early-stage glottic cancer. *Head Neck* 2016 Apr
- Monnier et al. Transthyroid access to the larynx for endoscopic resection of early-stage glottic cancer. *Head Neck* 2016 Apr
- Pediatric cases:**
 - Escriba Alepuz, Francisco J; Alonso Garcia, Javier; Cuchillo Sastriques, J. Vicente; Alcalá, Emilio; Argente Navarro, Pilar. Emergency Ventilation of Infant Subglottic Stenosis Through Small Gauge Lumen Using the Ventrain®. *Anesthesia & Analgesia* 2017
- Ventilation during lung separation procedures:**
 - V.M. Evers, R.V. Immink, W.J.P. van Boven, M.I. van Berge Henegouwen, M.W. Hollmann and D.P. Veelo. Intraoperative Use of the Ventrain for Single Lung Ventilation After Iatrogenic Trauma to the Left Main Bronchus During Thoracoscopy: A Case Report. *A&A Case Reports*. Apr 2017 epub
 - P. A. J. Borg*, A. E. W. Hamaekers, M. Lacko, J. Jansen D. Enk. Ventrain for ventilation of the lungs. *Br. J. Anaesth.* (2012) 109 (5): 833-834.
 - Response to "Ventrain for ventilation of the lungs". Fauzia A MirAnil Patel, Catriona Ferguson Royal National Throat, Nose and Ear Hospital, London. *Br. J. Anaesth.* (December 18, 2012).
 - Hamaekers AEW, Borg PAJ, Enk D. Ventrain: an ejector ventilator for emergency use. *Br J Anaesth.* 2012;108:1017-21
 - Kalkoff M, Shetty S. Ventilation through a small-bore intubating catheter using Ventrain in an elective procedure. *Br. J. Anaesth.* (18 December 2012).
 - Kalsi et al. Transtracheal jet ventilation using the Ventrain Device in a patient with severe upper airway obstruction. *Poster DAS 2012*
 - Dias et al. Adequate minute volume ventilation through a 100 cm long, 3 mm inner diameter airway exchange catheter by expiratory ventilation assistance (EVA). *Poster ESA 2012*
- Manoach et al. Resuscitation from prolonged apnea and upper airway obstruction in a large ovine model: a pilot trial of transtracheal ventilation using 15 lpm with and without an entrainment assisted trans-catheter exhalation device. *Poster SAM 2011*
- Hamaekers A, Theunissen M, Jansen J, Honig W, Enk D. Fast recovery from severe hypoxia through a 2 mm transtracheal catheter in pigs with a completely obstructed upper airway. *Eur J Anaesthesiol* 2010;27(Suppl. 47):19AP, 1-9.
- R. R. Noppens. Ventilation through a 'straw': the final answer in a totally closed upper airway? *Br. J. Anaesth.* (2015) 115 (2): 168-170.
- Hamaeker AEW, van der Beek T, Theunissen M, et al. Ventilation Through a Small-Bore Transtracheal Cannula in Severe Hypoxic Pigs Using Expiratory Ventilation Assistance. *Anesth Analg*. 2015 Apr; 120(4): 890-894.
- Paxian M, Preussler NP, Reinz T, Schlueter A, Gottschall R. Transtracheal ventilation with a novel ejector-based device (Ventrain) in open, partly obstructed, or totally closed upper airways in pigs. *Br. J. Anaesth.* 2015 Aug;115(2):308-16.
- Willemsen M.G.A., Noppens R, Mulder A.L.M., Enk D. Ventilation with the Ventrain through a small lumen catheter in the failed paediatric airway: two case reports. *Br. J. Anaesth.* (2014) 112 (5): 946-947.
- Fearnley et al. Use of the Ventrain in a patient with complete upper airway obstruction preventing expiration during high frequency jet ventilation. *Poster DAS 2014*
- Berry et al. Percutaneous transtracheal ventilation in an obstructed airway model in post apneic sheep. *Br. J. Anaesth.* 2014
- P. Nellgård. Ventrain in a case of can't intubate can't ventilate situation. *Abstract ESA 2013*
- De Wolf et al. Emergency ventilation through an airway exchange catheter using Ventrain: an animal study. *Poster DAS 2013*
- Calderon et al. Expiratory ventilation assistance (EVA) through a 14G catheter (2mm) in a Totally Obstructed Airway (TOA). *Poster SAM 2013*

ORDER INFORMATION

PRODUCT	REF
Ventrain Emergency Kit: Ventrain® + Cricath® 	#2618300
Ventrain® 	#2618200
Tritube® 	#7000
Manometer 	#AQ0012

Ventrain®

Redefining Ventilation

Contact IES Medical®

International Emergency Services, S.L.
 Bizkaia Science and Technology Park
 Ibaizabal bidea, 500
 ES 48160 Derio (Bizkaia), Spain
 Tel + 34 644 368 922
 email: info@ventinovamedical.com
 www.iesmedical.es

Contact Ventinova Academy

Risseweg 13,
 6004 RM Weert, The Netherlands
 Tel: +31 649 998 203
 www.ventinovamedical.com



CAT_VENTRAIN_EN_03