

» Literature List

Clinical use, pre-clinical validation and technological development of our products have been published. Also the products of Ventinova Evone®, Ventrain®, Tritube® and Cricath® are included in various handbooks. Publications regarding FCV® or EVA® ventilation during General Surgery, Intensive Care, Upper Airway Surgery, Lung Separation Procedures, Emergency or regarding Technology Development are presented.


Specific features of FCV® and EVA® described are indicated:



Small Lumen



Higher Efficiency

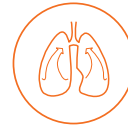

Lower Energy

))) **evone**

Upper Airway Surgery

Evone – Clinical Cases – Upper Airway Surgery

Dr. Imran Ahmad is the first to describe a challenging airway case, in which Tritube and Evone were found of significant value. The patient, scheduled for panendoscopy, had an anticipated difficult airway combined with airway pathology and COPD. Dr. Ahmad therefore opted for awake tracheal intubation with Tritube. After awake placement of the flexible bronchoscope, Tritube was guided using a silk suture tied over the bronchoscope. The patient was anesthetized and adequately ventilated with Evone for 45 minutes. Tritube allowed adequate surgical access with the advantage of a definitive airway, whilst continuous ventilation was delivered.



K. Jeyarajah and I. Ahmad. Awake tracheal placement of the Tritube under flexible bronchoscopic guidance. Anaesthesia Cases / 2018-0097 / ISSN 2396-8397 epub Jul 2018

General Surgery

Evone – In-Vivo Study – General Surgery

The workgroup of Prof. Schumann was the first to execute a comparative study, evaluating a commonly applied VCV protocol with FCV[®] mode by Evone. This study in healthy pigs shows that FCV[®] improves lung aeration via elevated mean tracheal pressure and consequently improves arterial oxygenation (+10%, P=0.002) at unaltered positive end-expiratory pressure (PEEP) and peak inspiratory pressure (PIP), while using a lower minute volume (-21%, P=0.04). Moreover, these findings suggest the FCV[®] mode provided by Evone is a new approach for protective lung ventilation.

J. Schmidt, C. Wenzel, M. Mahn, S. Spassov, H.C. Schmitz, S. Borgmann, Z. Lin, J. Haberstroh, S. Meckel, S. Eiden, S. Wirth, H. Buerkle and S. Schumann. Improved lung recruitment and oxygenation during mandatory ventilation with a new expiratory ventilation assistance device: A controlled interventional trial in healthy pigs. Eur J Anaesthesiol. 2018 35:1-9



Intensive Care

Evone – In-Vivo Study – Intensive Care

The workgroup of Prof. Schumann demonstrated in porcine ARDS, that FCV[®] ventilation stabilizes alveolar walls and shifts ventilation distribution from atelectatic to aerated lung tissue. Consequently, FCV[®] ventilation improves oxygenation (+53%, P=0.04) and facilitates CO₂ removal with a lower Minute Volume (-30%; P<0.001). This study illuminates the higher ventilation efficiency and lung protective potency of FCV[®] in ARDS.

J. Schmidt, C. Wenzel, S. Spassov, S. Wirth and S. Schumann. Expiratory Ventilation Assistance during mandatory ventilation in porcine ARDS improves arterial oxygenation – a randomized controlled animal study. Abstract Euroanaesthesia 2018



FCV[®] and Lower Energy

Evone – Theoretical Evidence – FCV[®] and Lower Energy

Prof. Barnes, together with Van Asseldonk and Enk, provide clear theoretical evidence for lower energy dissipation in the lungs by FCV[®] as compared to VCV or PCV. They present a simple analysis and numerical calculations indicating that energy dissipation may be



substantially reduced by controlling the ventilation flow to be constant and continuously during both inspiration and expiration and by ventilating at an I:E ratio very close to 1:1 – that is by using FCV®.

T. Barnes, D. van Asseldonk and D. Enk. Minimisation of dissipated energy in the airways during mechanical ventilation by using constant inspiratory and expiratory flows – flow controlled ventilation. Medical Hypotheses 2018 conditionally accepted

Evone – Clinical Proof of Concept – FCV® and Lower Energy

Profs Barnes and Enk are the first to actually determine pressure-volume (PV) loops and to show minimized energy dissipation in a patient ventilated with FCV® by Evone. During FCV® ventilation, both inspiratory and expiratory flows were kept nearly constant around 12 L/min and the I:E ratio was 1:1 resulting in a minute volume of 6.2 L/min. PV loops were recorded using pressure measured directly within the patient's trachea and the energy dissipated in the patient was calculated from the hysteresis area of the PV loops. The energy dissipation was 0.17 J/L, which is even lower than values quoted in literature for spontaneous breathing (0.2-0.7 J/L). They state that FCV® may have implications for lung-protective ventilation.

T. Barnes and D. Enk. Ventilation for low dissipated energy achieved using flow control during both inspiration and expiration. Trends in Anaesthesia and Critical Care 2018, in press



› ventrain ◁

This review by Dr. Doyle provides a nice overview of currently available literature regarding the Ventrain ventilation system. A short explanation of the physical principles underlying the functional concept of Ventrain is followed by a summary of bench and animal studies that demonstrate novelty and efficient performance of the device. Dr. Doyle then highlights clinical cases and studies demonstrating clear advantages of Ventrain compared to conventional ventilation techniques through small lumen catheters. He describes the use of Ventrain during upper airway surgery, in emergency CICO ('Cannot Intubate, Cannot Oxygenate')



situations and for critical pediatric airways. In sum, all published data support an emerging and promising role of Ventrain in clinical airway management.

D.J. Doyle. Ventilation via Narrow-Bore Catheters: Clinical and Technical Perspectives on the Ventrain Ventilation System. The Open Anaesthesia Journal 2018, in press

Upper Airway Surgery

Ventrain – Clinical Cases – Upper Airway Surgery

Dr. Kristensen and colleagues were the first to describe Tritube's clinical use in seven adult Ear-Nose-Throat surgical patients with airway narrowing or whose surgical access was facilitated by this small-bore endotracheal tube. In combination with Ventrain, adequate ventilation was achieved in all patients and intratracheal pressure was kept between 5 and 20 cm H₂O.

They concluded that: *"The 2.4 mm internal diameter Tritube seems to facilitate tracheal intubation and to provide unprecedented view of the intubated airway during oral, pharyngeal, laryngeal or tracheal procedures in adults."*

Additionally, they state that: *"This technique has the potential to replace temporary tracheostomy, jet-ventilation or extra-corporal membrane oxygenation in selected patients."*

M.S. Kristensen, M.W.P. de Wolf and L.S. Rasmussen. Ventilation via the 2.4 mm internal diameter Tritube® with cuff - new possibilities in airway management. Acta Anesthesiol. Scand. 2017 Jul; 61(6):580-589



Dr. Ahmad and colleagues present a case of a patient with severe upper airway obstruction undergoing surgical intervention, avoiding the need for tracheostomy. The patient sternly refused an awake elective tracheostomy or wide-bore cricothyroid cannula, so a 2-stage airway management technique was performed: an awake fiberoptic intubation with a small diameter endotracheal tube, followed by needle cricothyroidotomy with Cricath after anesthetic induction. Ventrain was used to adequately ventilate the patient for 75 minutes (SpO₂ 100%, PaCO₂ 46 mmHg). Post-operatively Cricath was left in situ for 24 hours. No complications occurred. The patient was discharged home 2 days later. View this elegant two-stage airway management technique: <http://links.lww.com/AACR/A133>

D.N. Onwochei, K. El-Boghdady and I. Ahmad. Two-Stage Technique Used to Manage Severe Upper Airway Obstruction and Avoid Surgical Tracheostomy: A Case Report. A A Pract 2018 Mar 1;10(5):118-120



Dr. Fearnley and colleagues reported the elective use of Ventrain in a patient with post radiation fibrosis that had previously prevented passive expiration during attempted high frequency jet ventilation. Ventrain was found simple and easy to use and provided perfectly adequate transtracheal ventilation for > 1 hour, allowing laser resection of the stenosis. They have used our transtracheal catheter Cricath, which was found kink resistant. Ventrain has now become their first choice device when emergency needle cricothyrotomy is performed.

R.A. Fearnley, S. Badiger, R. Oakley and I. Ahmad. Elective use of the Ventrain for upper airway obstruction during high frequency jet ventilation. J Clin Case Rep 2016 (6):2



Dr. Borg and colleagues reported the successful and uneventful elective use of Ventrain in combination with a transtracheal catheter (2 mm ID) with 20 min of adequate ventilation and oxygenation in a patient with partial obstruction of the laryngeal inlet.

P.A. Borg, A.E.W. Hamaekers, M. Lacko, J. Jansen and D. Enk. Ventrain for ventilation of the lungs. Br J Anaesth. 2012 Nov;109(5):833-4



Dr. Monnier and colleagues reported a new surgical technique in a patient with an early stage squamous cell carcinoma of the glottis with involvement of both vocal cords and impossible transoral access to the larynx, due to a medical history of head and neck radiation. While the larynx was accessed transthyrohyoidly, Ventrain was used for transtracheal ventilation during the first part of the surgery, followed by high frequency jet ventilation when the airway was patent. This setup proved to be extremely safe for securing the airway and allowed endoscopic resection of the tumor.

Y. Monnier, P. Schoettker, B. Morisod, C. Ikonomidis and C. Simon. Transthyrohyoid access to the larynx for endoscopic resection of early-stage glottic cancer. Head Neck 2016 Aug;38(8):1286-9



Dr. Kalkoff described the first successful use of Ventrain in combination with an intubating catheter in an elective setting, in a patient undergoing microlaryngoscopy with a partly obstructed airway.

M. Kalkoff. Ventilation through a small-bore intubating catheter using Ventrain in an elective procedure. Letter Br J Anaesth 2012



In Ethiopia, Dr. Braga and colleagues electively used Ventrain in combination with a transtracheal catheter as a safe 'bridge to intubation' in a young patient presented for free flap surgery to cover a complex type IV NOMA defect.

B.A.J. Braga, G. Rodeny, G. Hotvedt, A.F.H. Taylor and D.R. Ball. Ventrain ejector ventilation as a bridge to tracheal intubation for complex NOMA pathology – Poster presented at the Difficult Airway Society 2014 and World Airway Management Meeting 2015



Ventrain – In-Vivo Studies – Upper Airway Surgery

The workgroup of Prof. Enk demonstrated with an early prototype of Tritube that small-bore ventilation with Ventrain is optimized in a cuffed airway. A pressure measurement line in the prototype Tritube enabled a reliable airway pressure monitoring using a cuff manometer. During the 30 minutes of ventilation with Ventrain (PaO₂ 61 [52-69] kpa; PaCO₂ 4.9 [4.2-6.2] kPa) hemodynamics were stable. This elegant study was executed in healthy pigs.

M. W. P. de Wolf, T. van der Beek, A. E. Hamaekers, M. Theunissen and D. Enk. A prototype small-bore ventilation catheter with a cuff: cuff inflation optimizes ventilation with the Ventrain. Acta Anaesth Scand 2018 Mar 62(3):328-335



Dr. Paxian and colleagues demonstrated that Ventrain can ensure sufficient oxygenation and ventilation through a small-bore transtracheal catheter in live pigs when the airway is open, partly obstructed, or completely closed. Additionally, the minute ventilation and avoidance of high airway pressures were superior in comparison with traditional hand-triggered jet ventilation, particularly in the event of complete upper airway obstruction.

M. Paxian, N.P. Preussler, T. Reinz, A. Schlueter and R. Gottschall. Transtracheal ventilation with a novel ejector-based device (Ventrain) in open, partly obstructed, or totally closed upper airways in pigs. Br J Anaesth 2015 Aug;115(2):308-16. (Also referred in section "Emergency")



Ventilation during lung separation procedures

Ventrain – Clinical Case – Ventilation during lung separation procedures

In a patient undergoing thoracoscopic esophagectomy and concomitant wedge resection, an iatrogenic lesion in the left main bronchus was observed following deflation of the right lung. Repair of the lesion required deflation of the bronchial cuff. This challenging situation was resolved by Dr. Evers. She used Ventrain to oxygenate the patient through an Arndt



Endobronchial Blocker through the lumen beyond the bronchial defect. With the use of this technique, oxygenation was maintained at an acceptable level during repair.

V.M. Evers, R.V. Immink, W.J.P. van Boven, M.I. van Berge Henegouwen, M.W. Hollmann and D.P. Veelo. Intraoperative Use of the Ventrain for Single Lung Ventilation After Iatrogenic Trauma to the Left Main Bronchus During Thoracoscopy: A Case Report. AA Case Rep 2017 Aug 15; 9(4):116-118

Emergency

Ventrain – Clinical Cases – Emergency

Dr. Gerling and colleagues saved a patient's life using Cricath and Ventrain. Deteriorating respiratory distress, increasing hypoxia, and decreasing level of consciousness of a transported patient forced a ground ambulance to stop at the emergency department of the Meander Medical Center Amersfoort. Upon arrival the patient presented in acute and severe respiratory distress due to near-complete upper airway obstruction with SpO₂ of 81%, which dropped to 37% within 5 minutes after arrival. Active ventilation was not possible. Quick intraoral inspection and fiber optic evaluation revealed massive edema, secretions and no airway. Cricath was placed, Ventrain was connected and ventilation was started. Peripheral oxygen saturation rapidly improved within 90 seconds to 99% and hemodynamics improved. The patient was manually ventilated with Ventrain for nearly 60 minutes before semi elective surgical tracheotomy was safely performed. No hemodynamic or ventilation abnormalities were observed during the use of the Ventrain.

D.A. Heuveling, H.F. Mahieu, H.G. Jongsma-van Netten and V. Gerling. Transtracheal Use of the CriCath Cannula in Combination With the Ventrain Device for Prevention of Hypoxic Arrest due to Severe Upper Airway Obstruction: A Case Report. A&A Practice July 2018 epub



Dr. Escribá presents the rescue of a difficult airway in a pediatric patient with subglottic stenosis with Ventrain. When the patient desaturated, the device enabled immediate ventilation during airway assessment through a rigid bronchoscope and restoration of normal oxygen saturation. Then, post intubation, ventilation with the Ventrain was valuable again, when conventional mechanical PICU ventilation was very difficult due to elevated pressures. This case clearly indicates that the new ventilation device Ventrain offers advantages over devices available until now.



F.J. Escribá Alepuz, J. Alonso García, J.V. Cuchillo Sastriques, E. Alcalá and P. Argente Navarro. Emergency Ventilation of Infant Subglottic Stenosis Through Small Gauge Lumen Using the Ventrain®. A A Prac 2018 Mar 15;10(6):136-138

Dr. Wahlen and colleagues described a case in which Ventrain in combination with a tube exchanger was used to ventilate a patient with life-threatening tracheal stenosis. After a failed initial intubation with an endotracheal tube (ID 5.0 mm) the tube exchanger was inserted allowing adequate ventilation with Ventrain until surgical tracheostomy was performed. Hemodynamic stability indicated that the active expiration induced by Ventrain prevented intrapulmonary pressure build-up by air trapping and subsequent barotrauma, which may be observed during traditional jet ventilation in a similar situation

B.M. Wahlen, H. Al-Thani and A. El-Menyar. Ventrain: from theory to practice. Bridging until re-tracheostomy BJM Case Rep 2017 Aug 16; 2017



Dr. Willemsen and colleagues reported cases of ventilation through a small lumen intubating catheter (35 cm long; 1.6 mm ID) and an exchange catheter (45 cm long, 1.6 mm ID) using Ventrain to manage critical paediatric airways in babies of 2.1 kg and 4.3 kg, respectively.

M.G. Willemsen, R. Noppens, A.L. Mulder and D. Enk. Ventilation with the Ventrain through a small lumen catheter in the failed paediatric airway: two case reports. Br J Anaest 2014 May;112(5):946-7



Dr. J. Lopez-Torres and colleagues present four case reports of Ventrain use: laryngeal microsurgery, foreign body in the airway emergency, subglottic stenosis and assistance to guide extubation in Pierre-Robin syndrome. They indicate that laryngospasm, edema or anatomical distortion, combined with over-vigorous jet insufflation can result in air trapping with subsequent barotrauma and hemodynamic instability. They state that Ventrain is the only ventilation device that provides full ventilation for these situations.

J. Lopez-Torres, F.J. Esciba, J. Alonso, J.V. Cuchillo and P. Argente. Ventrain device for difficult or obstructed airway: 4 case report. Abstracts / Trends in Anesthesia and Critical Care 2017; 12:31



Dr. Krapf demonstrated feasibility of using Ventrain in combination with needle cricothyrotomy in a pre-hospital CICV setting under reanimation conditions.

M. Krapf, D. Gäumann, J. Jacquier and S. Graf. Cannot intubate, cannot ventilate: Beatmung über einen 2-mm-Katheter in der Präklinik. Notfallpraxis 2016 8(39); 792-794 (article in German)



Dr. Nellgård underlined the importance of performing an early cricothyrotomy in a situation of CICV. After several failed attempts to intubate a patient with instable angina pectoris scheduled for coronary artery bypass surgery, bag and mask ventilation became unsuccessful. A transtracheal catheter (2 mm ID) was placed and ventilation with Ventrain increased saturation from <50% to ~80%. Then, cricothyrotomy was successfully performed using the Melker Cricothyroidotomy 5. Surgery was postponed and percutaneous coronary intervention was performed instead.

P. Nellgård. Ventrain in a case of can't intubate can't ventilate situation. Orally presented at the European Society of Anaesthesiology 2013



Dr. Kalsi and colleagues reported for the first time the use of Ventrain in combination with a transtracheal catheter (2 mm ID) in an emergency scenario. They demonstrated adequate ventilation with saturation levels >98%.

A. Kalsi, K. Konieczny and M. Turner. Transtracheal jet ventilation using the Ventrain Device in a patient with severe upper airway obstruction. Poster presented at the Difficult Airway Society 2012



Ventrain – In-Vivo Studies – Emergency

Prof. Enk's workgroup demonstrated that Ventrain provided rapid reoxygenation and effective ventilation through a 100 cm long airway exchange catheter (ID 3 mm) in severe hypoxic pigs with an obstructed airway. This study clearly indicates potential clinical applicability and usefulness of Ventrain, not only in combination with short, transtracheal cannulas but also with long small lumen tubes/catheters when (re)intubation is difficult or has failed.

M.W.P. de Wolf, R. Gottschall, M.P. Preussler, M. Paxian and D. Enk. Emergency ventilation with the Ventrain through an airway exchange catheter in a porcine model of complete upper airway obstruction. Can J Anaesth. 2017 Jan;64(1):37-44



Dr. Paxian and colleagues demonstrated that Ventrain can ensure sufficient oxygenation and ventilation through a small-bore transtracheal catheter in live pigs when the airway is open, partly obstructed, or completely closed. Additionally, the minute ventilation and avoidance of high airway pressures were superior in comparison with traditional hand-triggered jet ventilation, particularly in the event of complete upper airway obstruction.

M. Paxian, N.P. Preussler, T. Reinz, A. Schlueter and R. Gottschall. Transtracheal ventilation with a novel ejector-based device (Ventrain) in open, partly obstructed, or totally closed upper airways in pigs. Br J Anaesth 2015 Aug;115(2):308-16 (Also referred in section "Upper airway surgery")



The workgroup of Prof. Enk showed quickly restored oxygenation after ventilation with Expiratory Ventilation Assistance (EVA®) in cases of a completely or partially obstructed upper airway in severe hypoxic pigs. Reoxygenation and ventilation were less efficient when the upper airway was completely unobstructed.

A.E.W. Hamaekers, T. van der Beek, M. Theunissen and D. Enk. Rescue ventilation through a small-bore transtracheal cannula in severe hypoxic pigs using expiratory ventilation assistance. Anesth Analg. 2015 Apr;120(4):890-4



In post-apnoeic sheep, Dr. Berry and colleagues demonstrated that Ventrain provided stable oxygenation and effective ventilation at low airway pressures during emergency percutaneous transtracheal ventilation in critically obstructed airways. Manujet provided effective temporizing oxygenation in this situation with hypoventilation necessary to minimize barotrauma risk.

M. Berry, Y. Tzeng and C. Marsland. Percutaneous transtracheal ventilation in an obstructed airway model in post apnoeic sheep. Br J Anaesth. 2014 Dec;113(6):1039-45



Dr. Ziebart and colleagues confirmed in a pig model with upper airway obstruction adequate ventilation using Ventrain in combination with a transtracheal catheter. Furthermore, they underline the importance of training and education of users and adherence to the instructions for use in order to use Ventrain safely.

A. Ziebart, A. Garcia-Bardon, J. Kamuf, R. Thomas, T. Liu, A. Schad, B. Duenges, M. David and EK. Hartmann. Pulmonary effects of expiratory-assisted small-lumen ventilation during upper airway obstruction in pigs. Anaesthesia. 2015 Oct;70(10):1171-9



In a small pilot study the workgroup of Prof. Rosenblatt demonstrated that Ventrain rapidly corrected hypoxemia in a large ovine model.

S. Manoach, L. Paladino and W. Rosenblatt. Resuscitation from prolonged apnea and upper airway obstruction in a large ovine model: a pilot trial of transtracheal ventilation using 15 lpm with and without an entrainment assisted trans-catheter exhalation device. Poster presented at the Society for Airway Management 2011



Technology Development

Full articles – Technology Development

In a bench study Dr. Wirth and colleagues showed that active expiration assistance (using Ventrain) provided better maintenance of minute ventilation without intrinsic PEEP compared to conventional mechanical ventilation (passive expiration), when using a small endotracheal tube or cricothyrotomy catheter.

S. Wirth, L. Seywert, J. Spaeth and S. Schumann. Compensating Artificial Airway Resistance via Active Expiration Assistance. Respir Care. 2016 Dec;61(12):1597-1604



In a bench study the workgroup of Prof. Enk tested the efficacy of a prototype of Ventrain. Results of this study suggested that Ventrain is capable of achieving a normal minute volume for an average adult through a 2 mm ID transtracheal catheter.

A.E.W. Hamaekers, P.A. Borg and D. Enk. Ventrain: an ejector ventilator for emergency use. Br J Anaesth. 2012 Jun;108(6):1017-21.



In a bench study Dr. Schmidt and colleagues show that not every oxygen delivery device will generate enough driving pressure to deliver a predictable flow through Ventrain and a 2 mm (ID) 75 mm long transtracheal catheter. They confirmed that while using the prescribed oxygen supply system with a pressure compensated flow regulator, flow and tidal volume were predictable enabling adequate ventilation through a small lumen.

A.R. Schmidt, K. Ruetzler, T. Haas, A. Schmitz and M. Weiss. Impact of oxygen sources on performance of the Ventrain® ventilation device in an in vitro set-up. Acta Anaesthesiol Scand. 2016 Feb;60(2):241-9. Rebuttal by de Wolf MW, Schutzer-Weissmann JM. Acta Anaesthesiol Scand. 2016 Nov;60(10):1477-1478.



Dr. Schmidt and colleagues showed that a Respiration Function Monitor is capable of monitoring ventilation with Ventrain, which could make its use even safer.

A.R. Schmidt, K. Ruetzler, T. Haas, A. Schmitz and M. Weiss. Monitoring der Tidalvolumina bei Verwendung des Ventrain® Notfallbeatmungsgerätes. Anaesthesist 65:514–520. Rebuttal by de Wolf, Gottschall and Enk. Anaesthesist 2017 · 66:207–208

A functional model of Ventrain, based on EVA® technology was build by Prof. Enk: the DE5. In laboratory tests the workgroup of Prof. Enk showed that the DE 5 is an optimized ventilation ejector suitable for applying expiratory ventilation assistance.

A.E.W. Hamaekers, P.A. Borg, T. Götz and D. Enk. Ventilation through a small-bore catheter: optimizing expiratory ventilation assistance. Br J Anaesth. 2011 Mar;106(3):403-9



The ineffectiveness and danger of using transtracheal jet ventilation in cases of complete upper airway obstruction motivated Prof. Enk to search for a better solution. Prof. Enk's workgroup transformed a small, industrial ejector into a simple, manual ventilator providing expiratory ventilation assistance (EVA®). They showed that EVA® shortened the expiration time and that a minute volume up to 6.6 L/min could be achieved through a 2 mm ID transtracheal catheter in a simulated obstructed airway.

A.E.W. Hamaekers, T. Götz, P.A. Borg and D. Enk. Achieving an adequate minute volume through a 2 mm transtracheal catheter in simulated upper airway obstruction using a modified industrial ejector. Br J Anaesth. 2010 Mar;104(3):382-6



The importance of flow and pressure release in jet ventilation devices was demonstrated by Prof. Enk and his co-workers in a bench study where three previously described self-assembled jet devices and the Oxygen Flow Modulator were tested. In case of complete upper airway obstruction the OFM provides sufficient flow and pressure release, whereas the self-assembled jet devices tested are inherently dangerous constructions.

A.E.W. Hamaekers, P.A. Borg, T. Götz and D. Enk. The importance of flow and pressure release in emergency jet ventilation devices. Paediatr Anaesth. 2009 May;19(5):452-7

To the reduce risk of air trapping and to increase oxygenation efficacy an emergency transtracheal ventilation device needs to allow both inspiration and expiration. In absence of such a device Prof. Enk and his co-workers determined the capability of two self-assembled, three-way stopcock based jet devices and the Oxygen Flow Modulator to function as a bidirectional airway in conjunction with a small lumen catheter.



A.E.W. Hamaekers, P.A. Borg and D. Enk. A bench study of ventilation via two self-assembled jet devices and the Oxygen Flow Modulator in simulated upper airway obstruction. Anaesthesia. 2009 Dec;64(12):1353-8

Posters – Technology Development

Dr. Heerdt and colleagues showed that a prototypical small automated ventilator based upon the EVA® principle could be used to generate a controlled period of negative EEP. This resulted in improved hemodynamics relative to positive pressure ventilation (PPV) during experimental hemorrhagic shock.

P.M. Heerdt, D.A. Berli, H. Peprah-Mensah and S. Manoach. Automated ventilation with a novel small device based upon expiratory ventilation assistance improves hemodynamics in hemorrhagic shock. Poster presented at the Military Health System Research Symposium 2015

In a bench study Dr. Calderon and colleagues compared passive expiration with EVA® using Ventrain in a Totally Obstructed Airway model. They demonstrated that, in contrast to passive expiration, EVA® maintained an acceptable Minute Volume, avoiding auto-PEEP.



L.G.M.B Calderon, M.M. Moreira, G.L. Emídio, E.P. Corrêa, M.A. Carvalho-Filho and R.G.G. Terzi. Expiratory ventilation assistance (EVA®) through a 14G catheter (2mm) in a Totally Obstructed Airway (TOA). Poster presented at Society for Airway Management 2013

The workgroup of Prof. Enk showed in a bench study that Ventrain is capable of achieving adequate minute volume ventilation through a 100 cm long, 3 mm ID Airway Exchange Catheter by applying EVA® in a simulated completely obstructed airway.



E.M. Dias, A.E.W. Hamaekers, P.A.J. Borg and D. Enk. Adequate minute volume ventilation through a 100 cm long, 3 mm inner diameter airway exchange catheter by expiratory ventilation assistance (EVA®). Poster presented at the European society of Anaesthesiology 2012

In this bench study Prof. Enk demonstrated that Ventrain in combination with a prototype cuffed jet ventilator catheter should allow adequate minute volume ventilation in adult patients.

D. Enk and S. Enk. Small Lumen Ventilation through a Cuffed Jet Ventilation Catheter: Efficiency and Safety. Poster presented at the Society for Airway Management 2012



Editorials, Correspondence

Dr. Noppens deliberated Ventrain's clinical significance:

"The idea and working principle behind the Ventrain are very promising and have the potential to change the way we look at ventilation through small-bore cannulae, not only in emergency situations."

"For the benefit of patients in critical situations, it will be necessary to train and educate as many anesthetists as possible in this novel ventilation technique, not only to avoid inappropriate use, but to allow its incorporation into routine airway management in the future."

This editorial refers to the study of Paxian and colleagues, which elegantly demonstrates that Ventrain ensures sufficient oxygenation and ventilation through a small-bore transtracheal catheter in cases of an open, partly obstructed, or completely closed airway.

R.R. Noppens. Ventilation through a 'straw': the final answer in a totally closed upper airway? Br J Anaesth. 2015 Aug; 115 (2): 168-170

Drs. Timmermann, Chrimes and Hagberg responded to the published 2015 Guidelines of the Difficult Airway Society (DAS), which endorse scalpel cricothyroidotomy as the sole method for emergency front-of-neck access:

"This sole recommendation of a scalpel technique may increase the psychological barriers to successful achievement of front-of-neck access in an appropriate time frame when a CICO situation occurs. Training in both (scalpel-based and cannula-based) techniques remains warranted, and the option to use either technique should continue to be advocated."

In addition to airway management techniques, they advised on ventilation techniques to be used after obtaining front-of-neck access. They recommend a.o. Ventrain:

"These provide, in contrast to a jet ventilator, an affordable, simple mechanism for oxygen insufflation via a cannula that can be connected to a standard oxygen outlet and allows for passive or even active expiration via the cannula between breaths. Techniques have been developed for use of these devices, intended to minimize the risk of complications from volutrauma and barotrauma, even when the upper airway is completely obstructed."

This editorial emphasizes the multiple options for saving a patient's life in an emergency CICO situation.

A. Timmermann, N. Chrimes and C.A. Hagberg. Need to consider human factors when determining first-line technique for emergency front-of-neck access. Br J Anaesth. 2016 Jul;117(1):5-7

Dr. Lang also responded to the DAS guidelines expressing his concerns that the DAS Plan D amount to 'wishful thinking' as it aims to avoid life-threatening consequences of barotrauma:

"It focuses on attaining and then maintaining proficiency using an invasive technique (i.e., surgical cricothyroidotomy) with which many anesthesiologists are not comfortable and do not have an opportunity to practice."

He underlines that Ventrain is based on concepts and skills familiar to most anesthesia practitioners and that it can be used in routine cases in order to gain clinical experience. As such the technique is trainable. Dr. Lang concludes:

"I also believe, most importantly, that awareness of, familiarity and proficiency with, and dissemination of the device will certainly be easier and quicker to achieve than a cultural shift that would be necessary to ensure maintaining competence with surgical cricothyroidotomy."

S.A. Lang. Emergency airway management: What are the roles for surgical cricothyroidotomy and the Ventrain® device? Can J Anaesth. 2016 Aug;63(8):997-8

Handbooks

- C. Hagberg. Benumof & Hagberg's Airway Management, 4th ed. Elsevier (publisher); 2018; p.106-107; p. 865-866.
- C. Hagberg. Benumof & Hagberg's Airway Management, 3rd ed. Elsevier (publisher); 2013; p.595-6.
- P.G. Barash, B.F. Cullen, R.K. Stoelting, M. Cahalan, M.C. Stock and R. Ortega. Clinical Anesthesia, 7th ed., Wolters Kluwer Health (publisher); 2013; p.798.
- C. Spies, M. Kastrup, T. Kerner, C. Melzer-Gartzke, H. Zielke and W.J. Kox. SOPs in Anästhesiologie und Schmerztherapie. Thieme (publisher); 2013; p.94.
- Dornberger. Schwieriges Atemwegsmanagement bei Erwachsenen und Kindern. Thieme (publisher); 2013; p.251-2.
- W. Rosenblatt and W. Popescu. Master techniques in Upper and Lower Airway Management. Wolters Kluwer Health (publisher); 2015; Chapter 44 (ePub).

Notes

» Adressess

Ventinova Medical B.V.

A Meerenakkerplein 7
5652 BJ Eindhoven
The Netherlands
T +31 (0)40 751 60 20
E info@ventinova.nl

Ventinova Deutschland GmbH

A Siemensstraße 31
47533 Kleve
Germany
T +49 (0)282 1975 02 73
E info@ventinova.de

MC012.06

FCV® is initially developed in it's most basic form as EVA® (Expiratory Ventilation Assistance)

© 2018 Ventinova, Eindhoven The Netherlands, all rights reserved.

® 2018 Ventinova, EVA, FCV, Evone, Tritube, Ventrain and Cricath are registered trademarks of Ventinova Medical.



www.ventinovamedical.com

